

Silent Knight Tablet Crusher



The dual action “crushing and grinding” movements of the Silent Knight Tablet Crusher result in silent, smooth, effortless and effective crushing of pills to fine powder.

The main advantages of this system are:

- Silent – will not disturb patients or staff
- Ergonomic Handle – fits all hand sizes
- Effective – powerful dual action crushes pills to powder in a few seconds
- Patient Safety – Eliminates cross contamination
- User Safety – Maximum results with minimal effort. Contour cover encloses internal moving parts
-

How it works in 3 easy steps



Place pill(s) into Silent Knight Pill Crusher Pouch. Lift handle and place pouch down between the crusher plates to initiate crushing.



Press handle down to crush pill(s). Lift handle and pull pouch upward to half way up the crusher plates.



Then press the handle down to activate grinding movement. Press handle down several times to grind pill(s) into fine powder. Repeat if needed. Then remove the pouch and add the medication to an appropriate serving medium for administration.